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EN

How to find the RECAcare tampon that suits you:


We have put together test sets for you. Test the small tampons first. If stool or the tampon is pushed out, please try the next size.

How to use the RECAcare tampon correctly:

- Wash your hands and remove the foil.
- Unwind the string from the tampon.
- **Soften RECAcare in warm water for one minute and squeeze out the tampon.**
- Place the RECAcare tampon on the insertion device.
- Get into a comfortable position (lie on your back or put one leg up, e.g. on a chair or the toilet).
- Insert the tampon into the rectum and when it is inside pull out the insertion device (when the sphincter is touched, a squeeze reflex may occur. In that case wait approximately 8 to 10 seconds until the reflex subsides and then insert the tampon).
- You can remove RECAcare by means of the string at any time during your visit to the toilet and dispose of it in the household waste.

Your well-being is important to us:

- If you want you can attach the string with a skin-friendly plaster onto the skin of one buttock.
- Remove the tampon latest after 20 hours.
- You do not need to remove the tampon when urinating.
- Nor does it need to be removed when swimming.
- Clean the insertion device after use with pH-neutral soap and rinse thoroughly with water.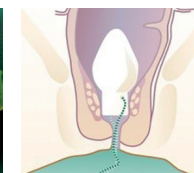

Contraindications:

- Bleedings of unknown origin
- Intestinal obstruction
- Acute inflammations in the abdominal area
- Unknown pains in abdominal cavity or perforation of the intestine

Side effects:

When using RECAcare, you may experience an occasional feeling of pressure. This feeling usually disappears after 3 to 4 days and certainly once the tampon has been removed.

Warning:

Please follow the instructions for use. RECAcare is a single-use product. RECA Med accepts no liability for any damage caused by improper use. In case of multiple use, the hygienic conditions may not be ensured anymore.